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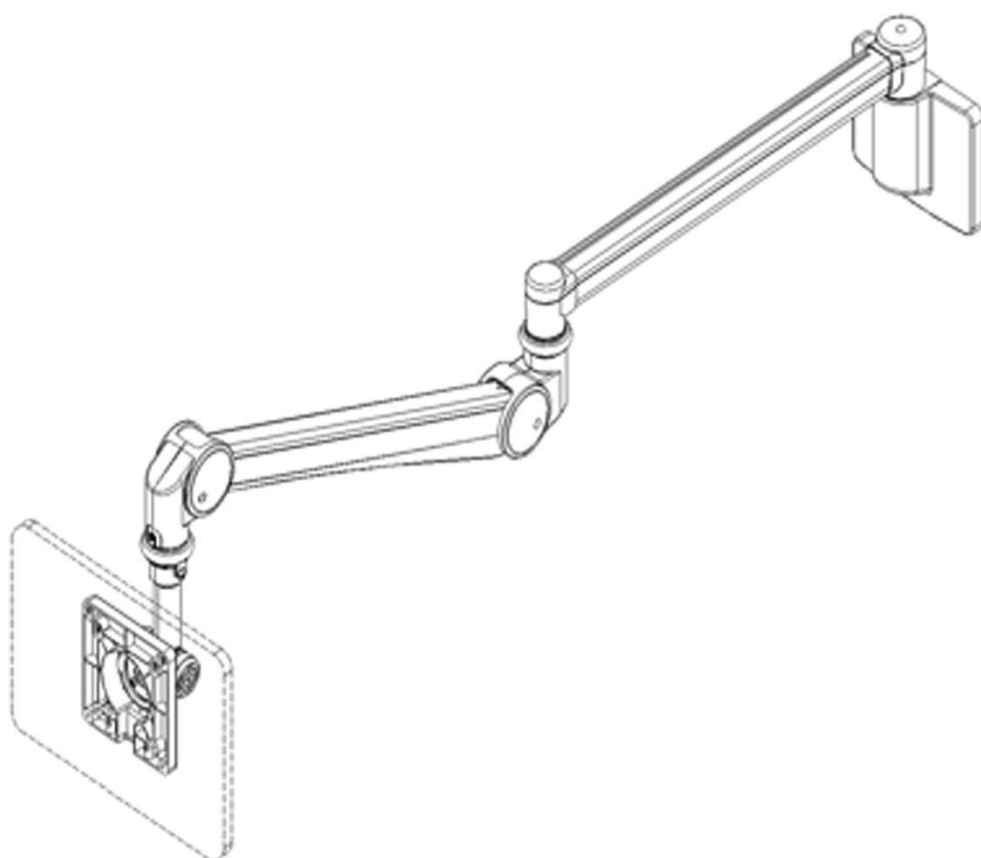
Installation manual

TMO-MED-029

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
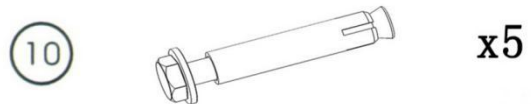
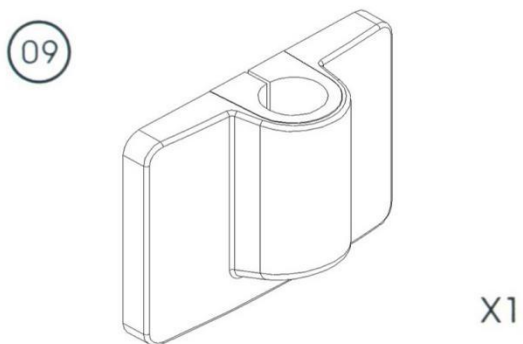
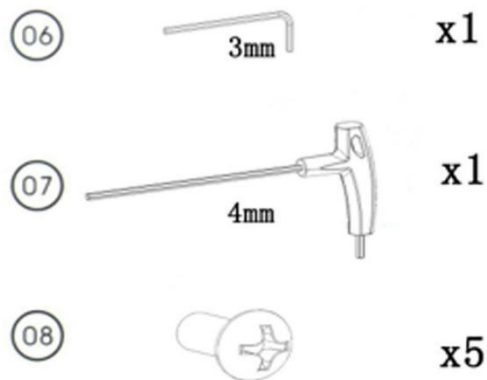
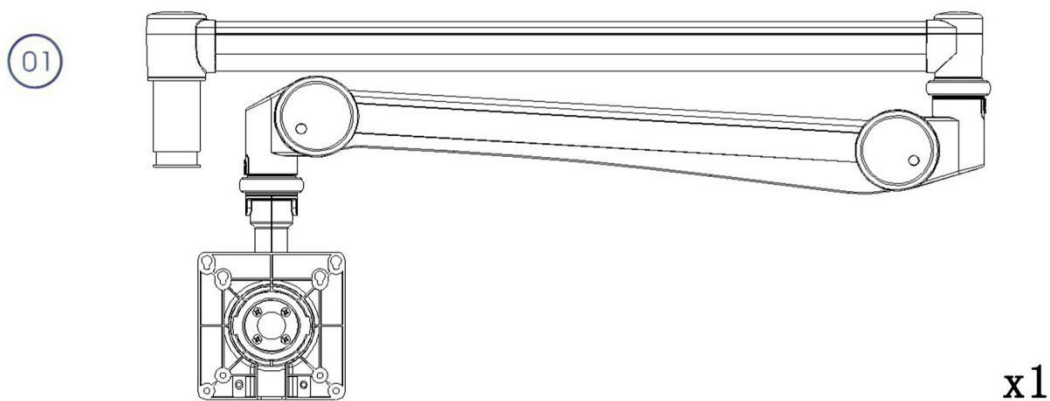
The installation should be performed by a qualified technician and should meet the requirements of local regulations.

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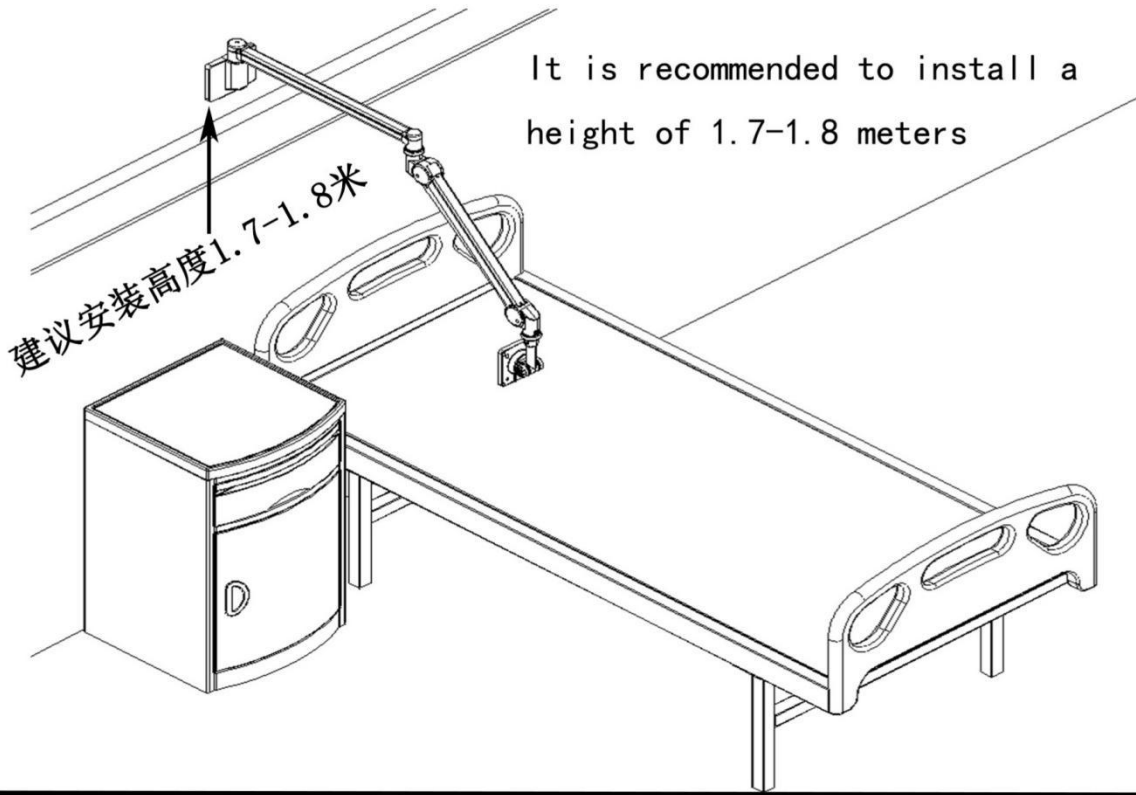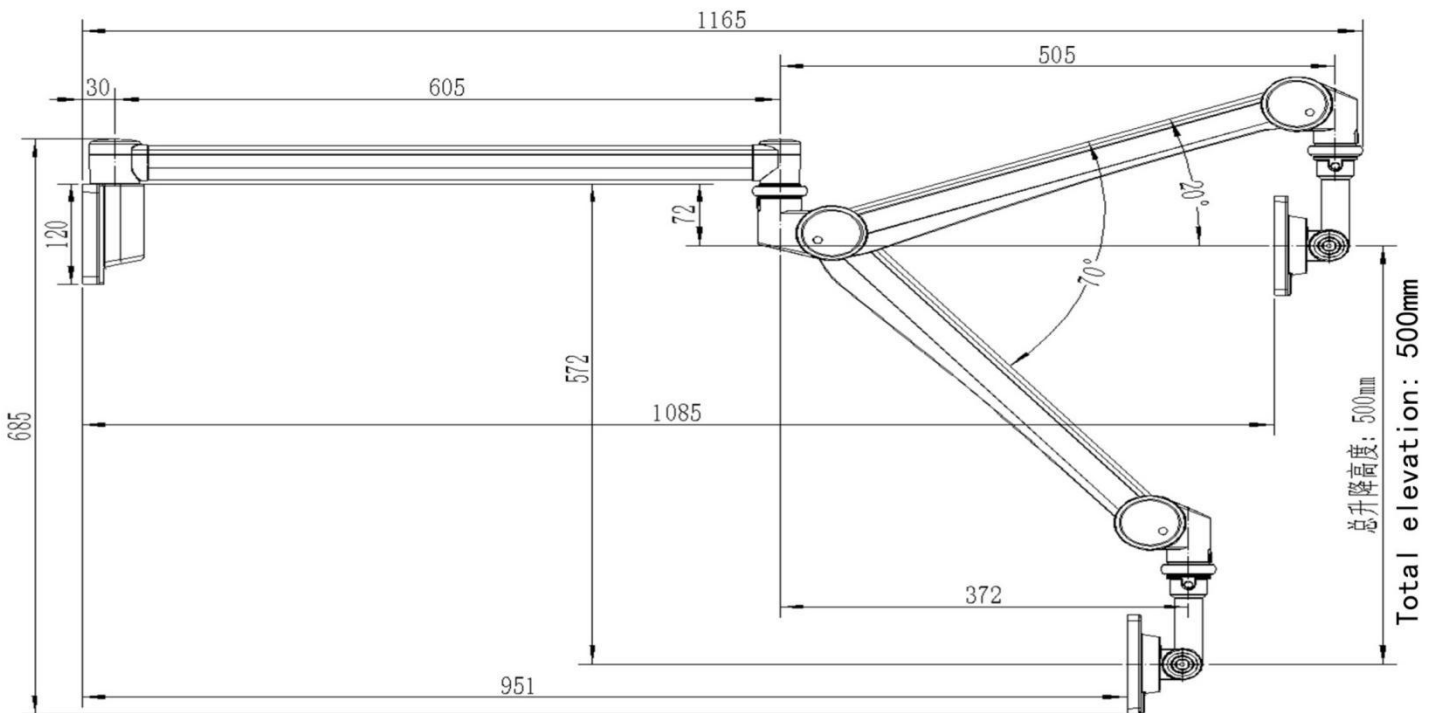
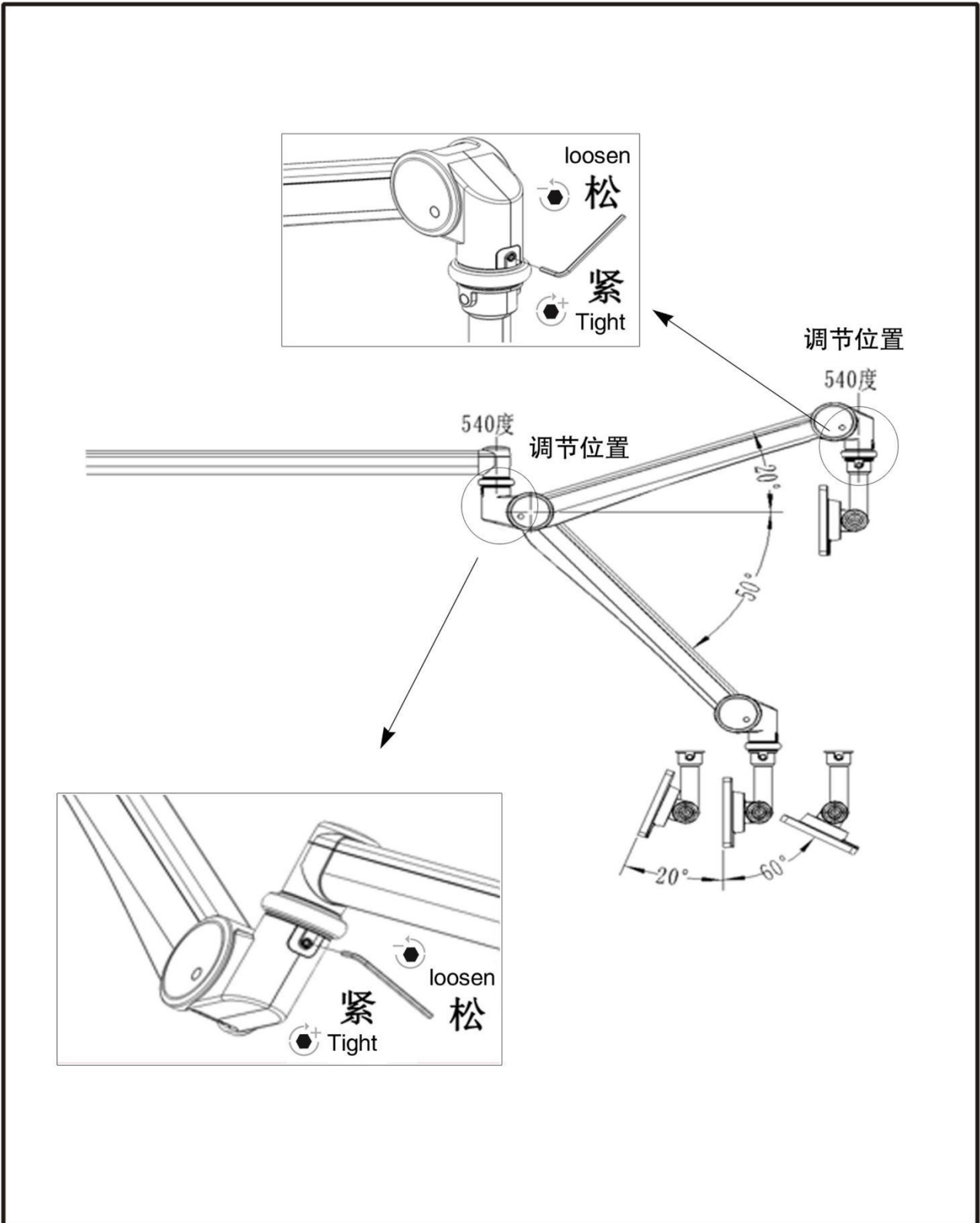
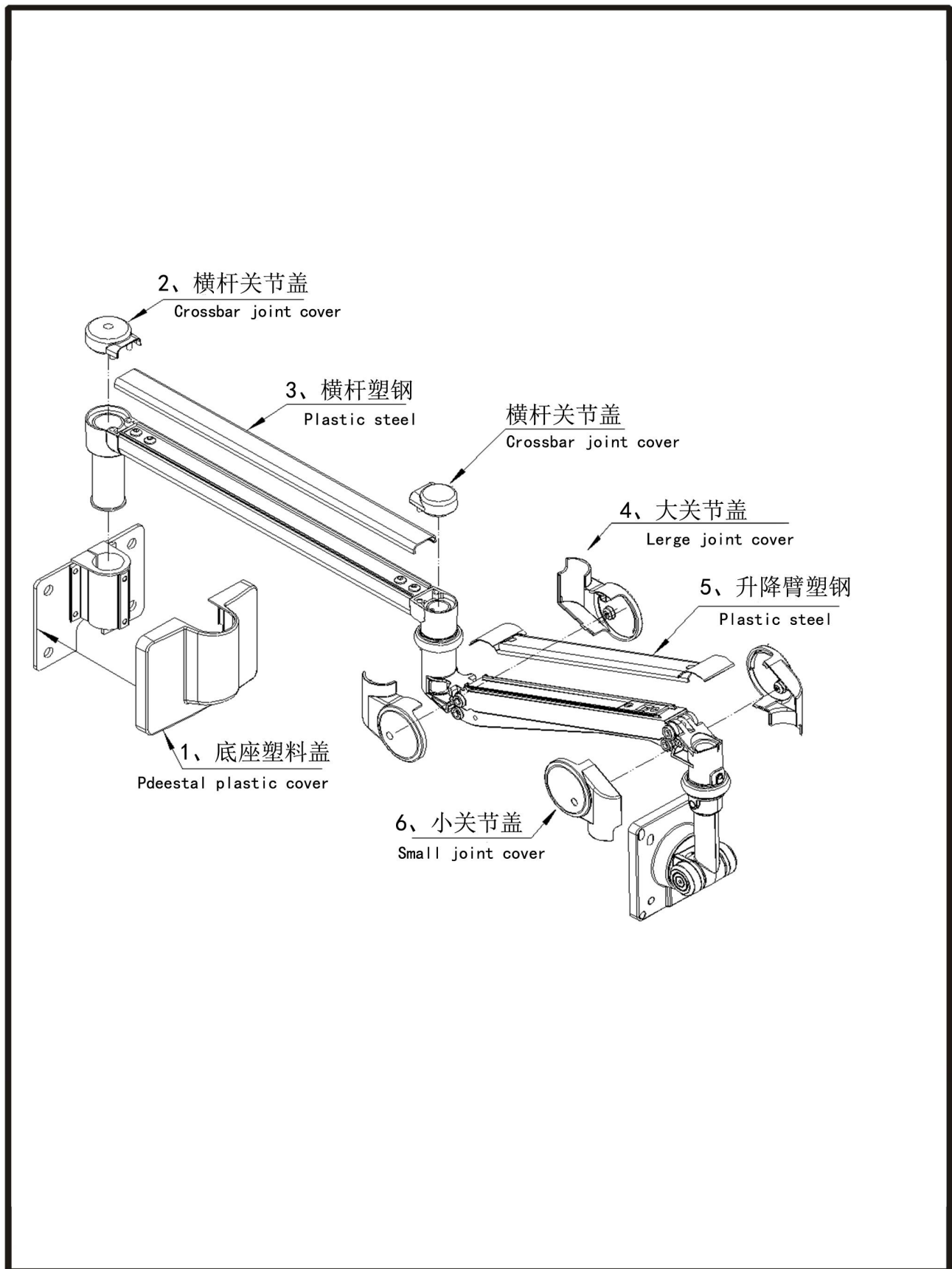
Please confirm the installation wall before assembly

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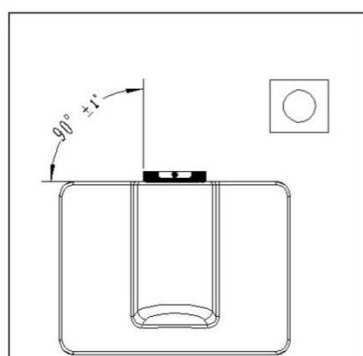
Put the base on the wall and mark four holes

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Drill 4 holes at a depth of 40mm with a 10mm drill

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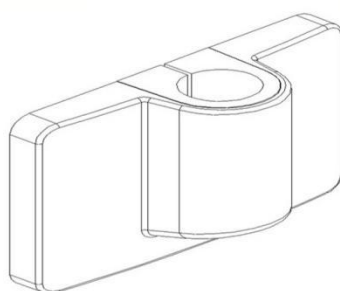
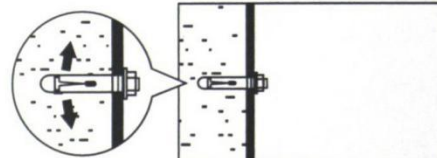
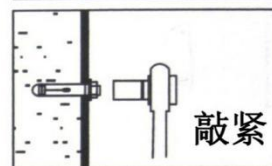
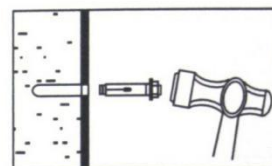
Use a level ruler to assist with the installation.

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


recommended drill bit size: 10mm / depth 40mm

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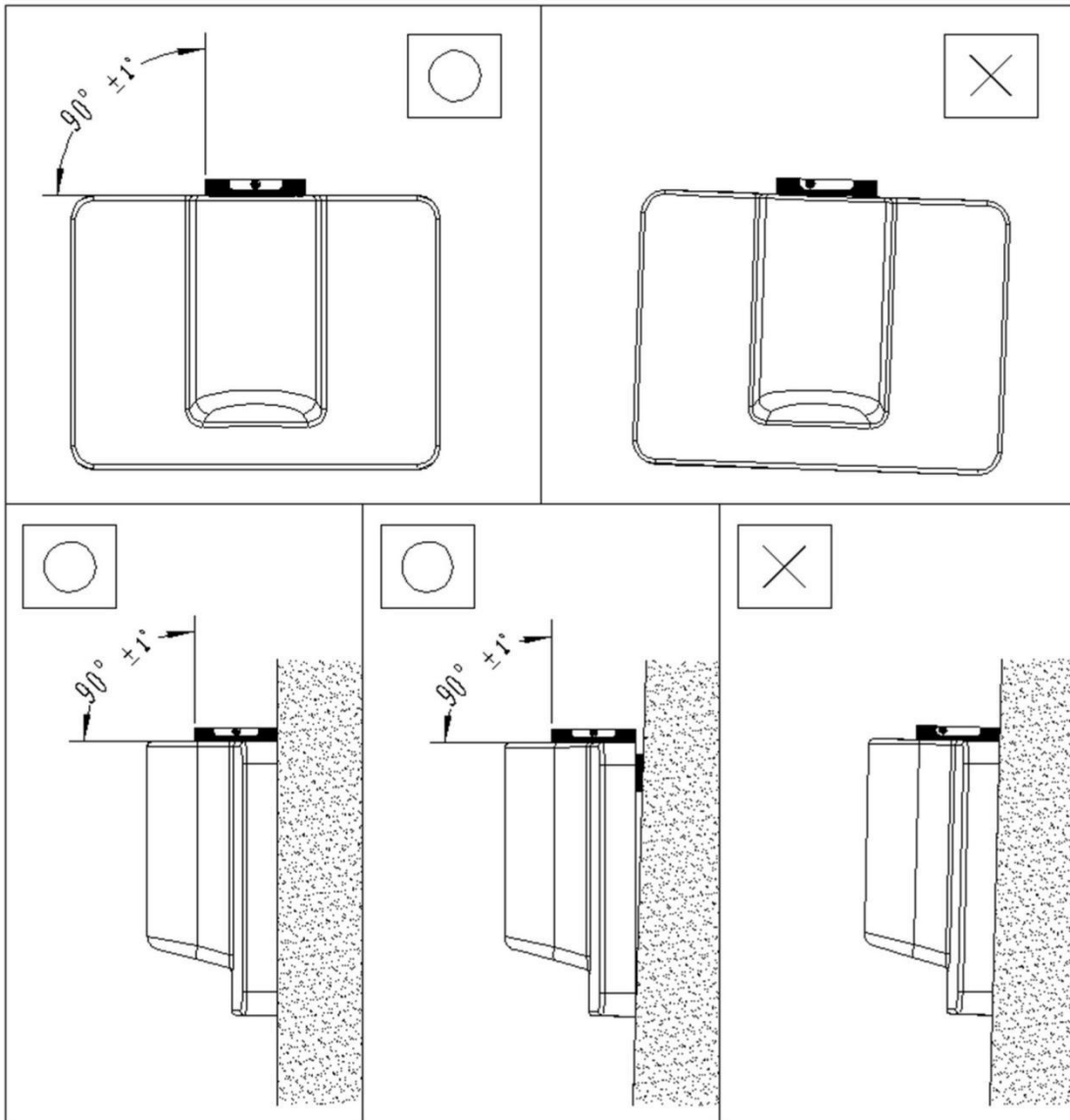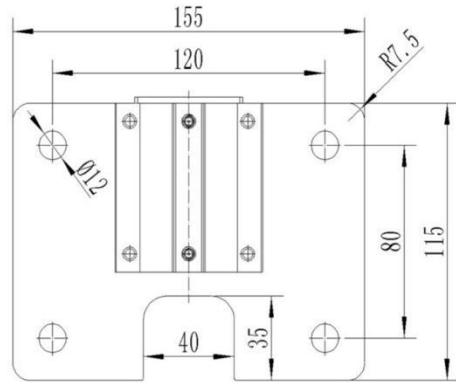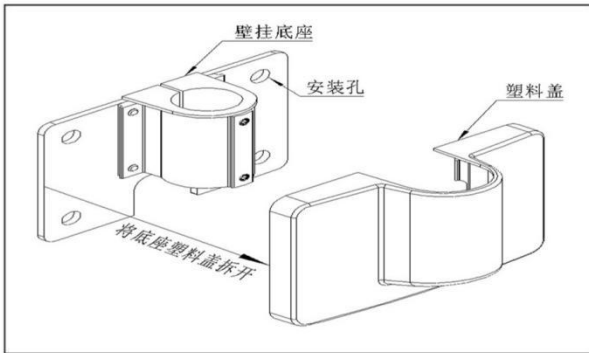## Installation instructions of the base

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


## Pedestal debugging

1.2

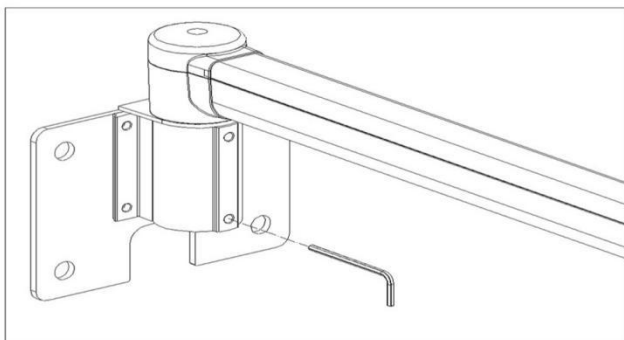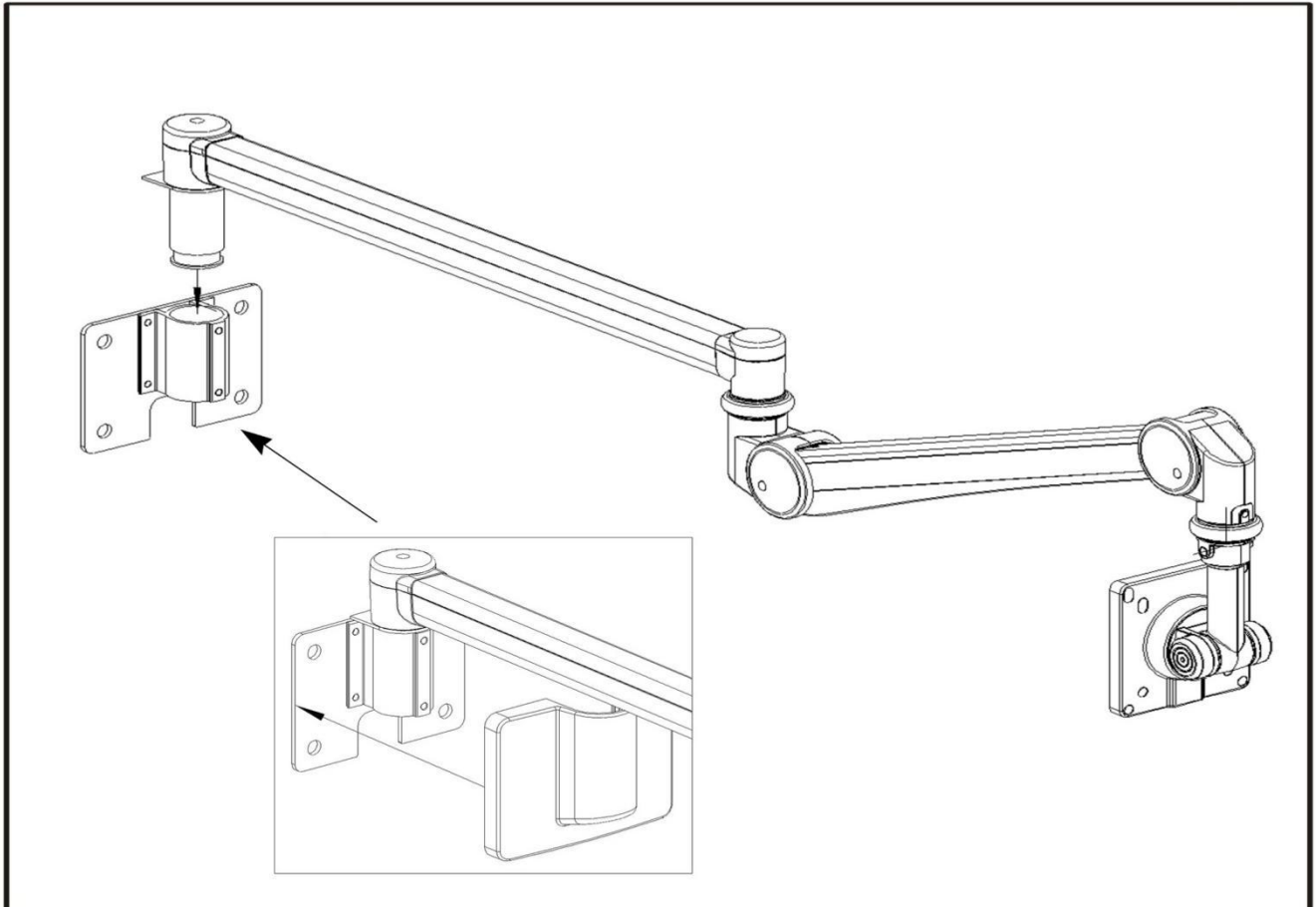用3mm六角把手调节壁挂式手臂放松这个螺丝，让手臂可以旋转  
Use the 3mm six knob to adjust the wall hanging arm to relax the screw so that the arm can be rotated.

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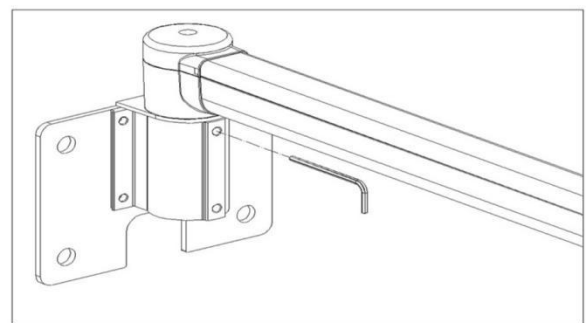
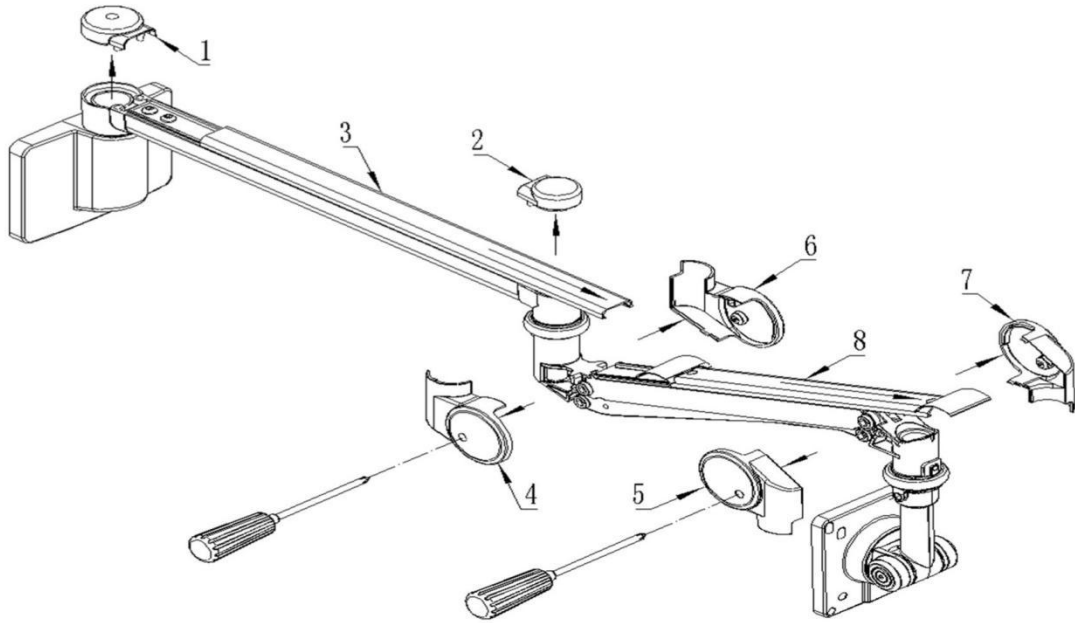
Disassemble according to numbered order

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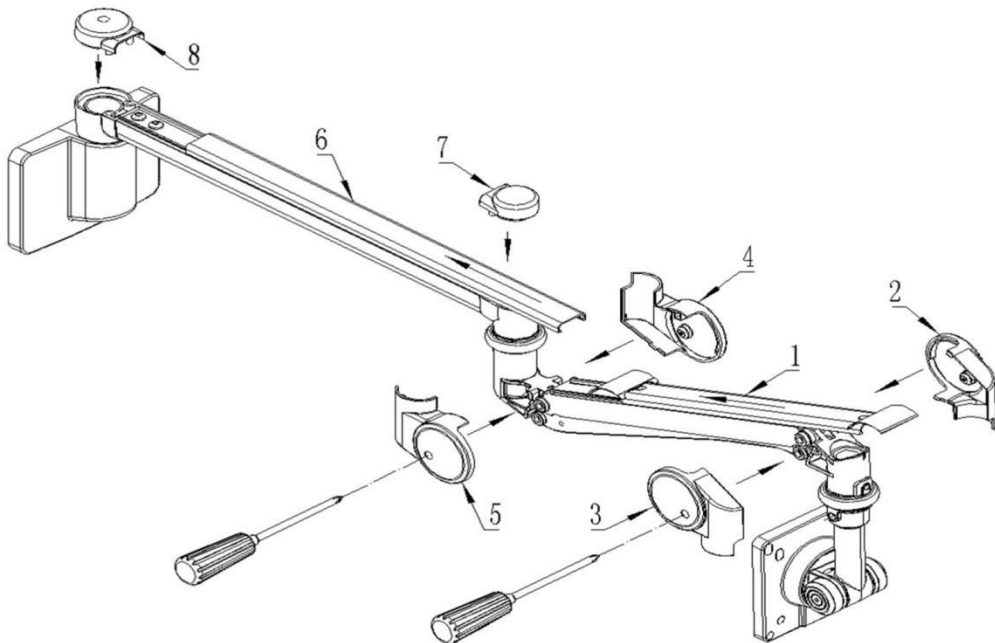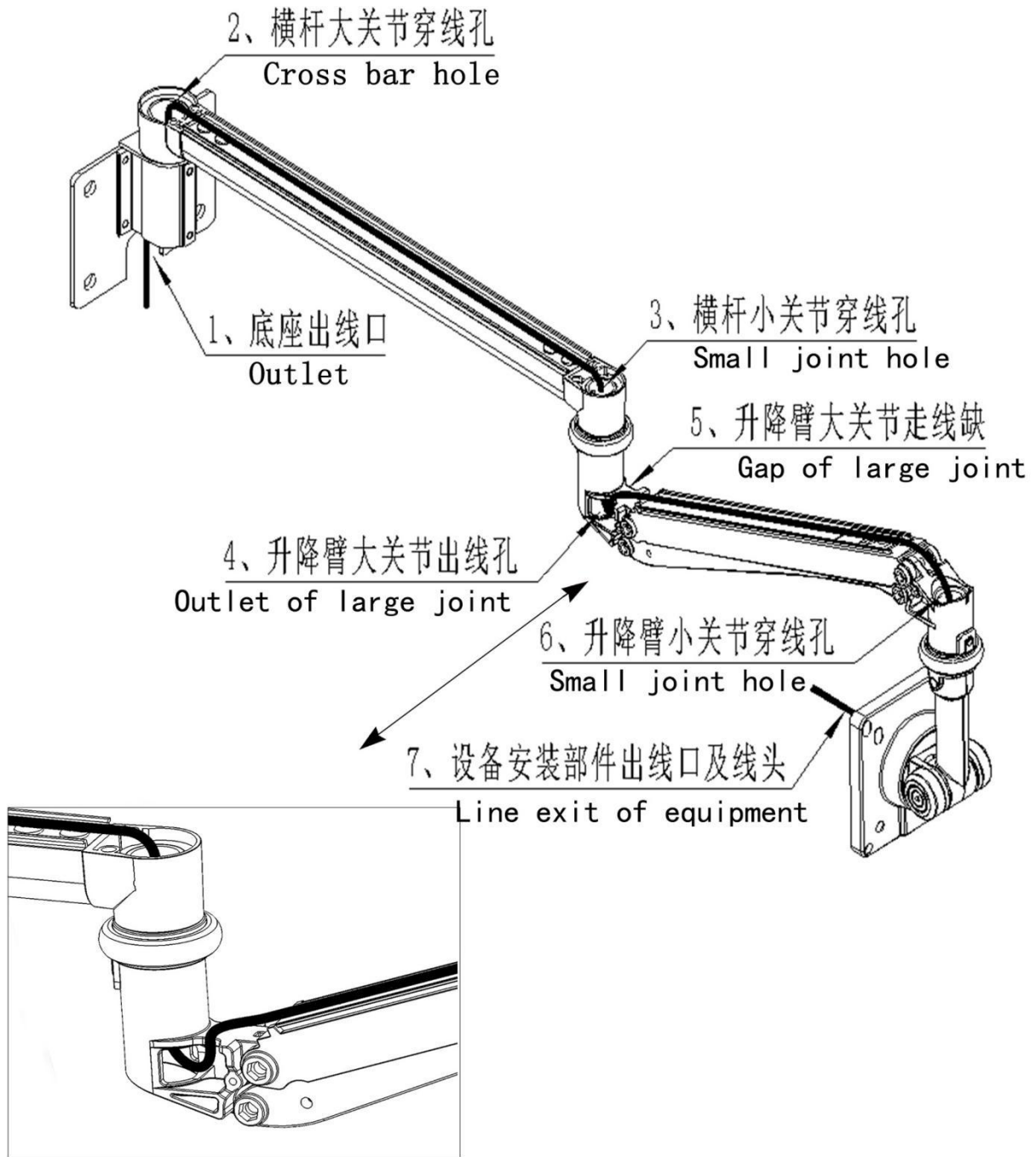
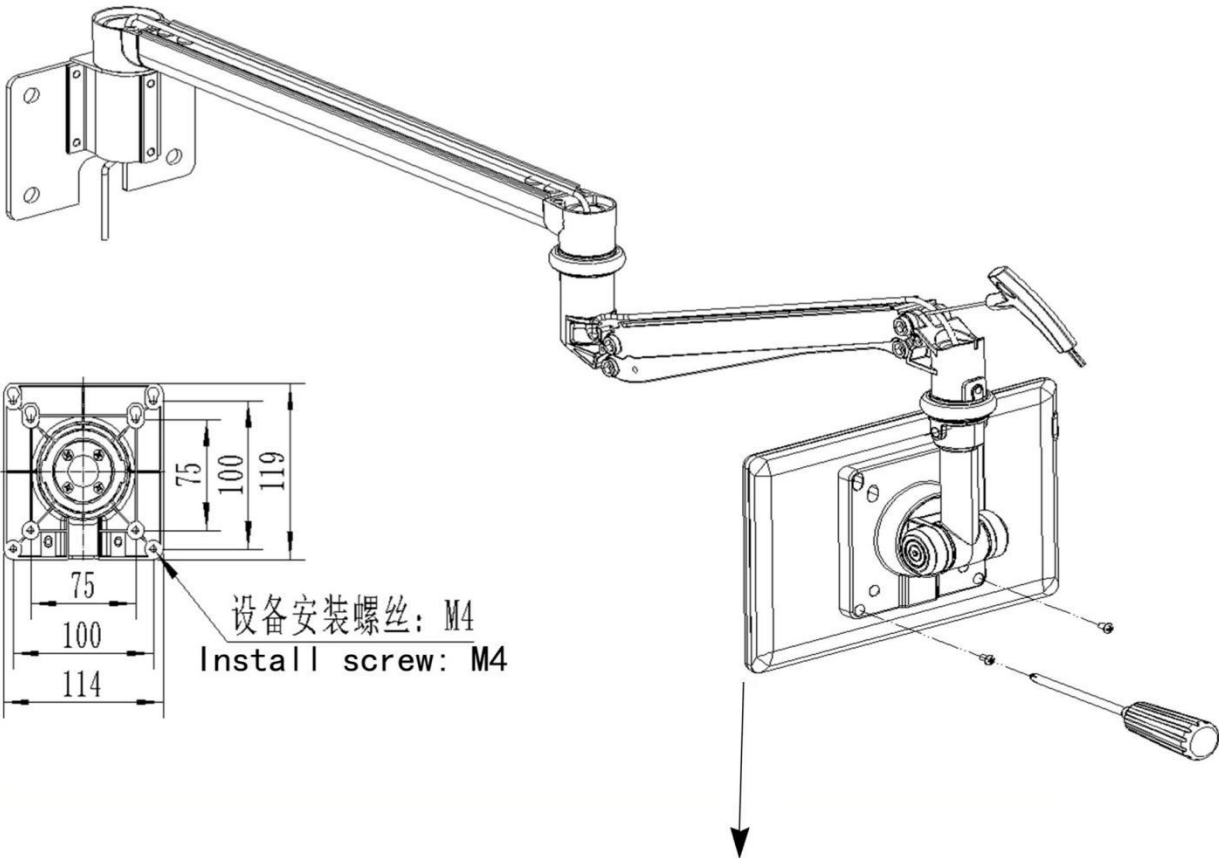
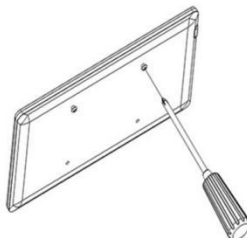
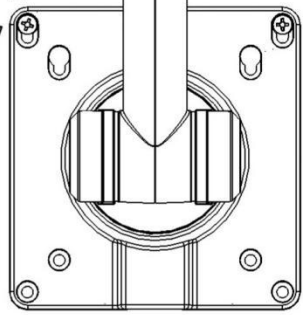
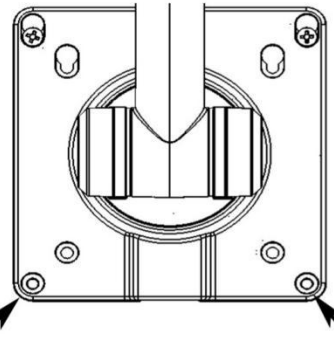
**1.1** 用螺丝刀安装显示器  
Mounting a display with a screwdriver.

 <p>1、在设备上先安装2颗螺丝 Install 2 screws first</p>	<p>下滑 Slide downward</p>  <p>2、将螺丝装进对应孔位快装口 Screw into fast loading port</p>	 <p>3、锁紧下面两个螺丝 Lock all the screws</p>
---	---	---

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## Adjusting load bearing

谨慎调节承重: Prudent regulation of load-bearing

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Make sure you know the weight of the equipment

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Support the arm throughout/during assembly

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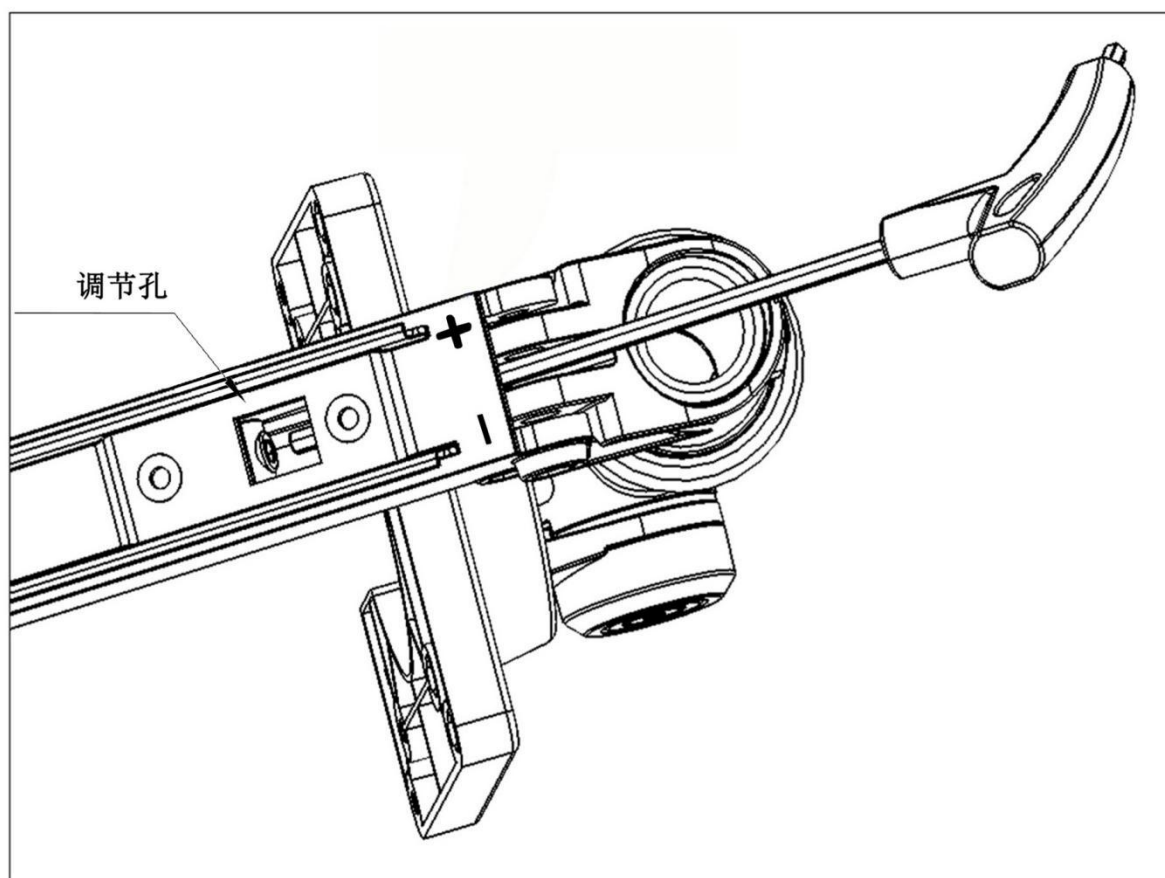
Use the six corners handle to adjust the load

increase clockwise (+) load-bearing

counterclockwise (-) reduce bearing.


承重范围: 0.5kg-2kg    1kg-4kg

Load-bearing range

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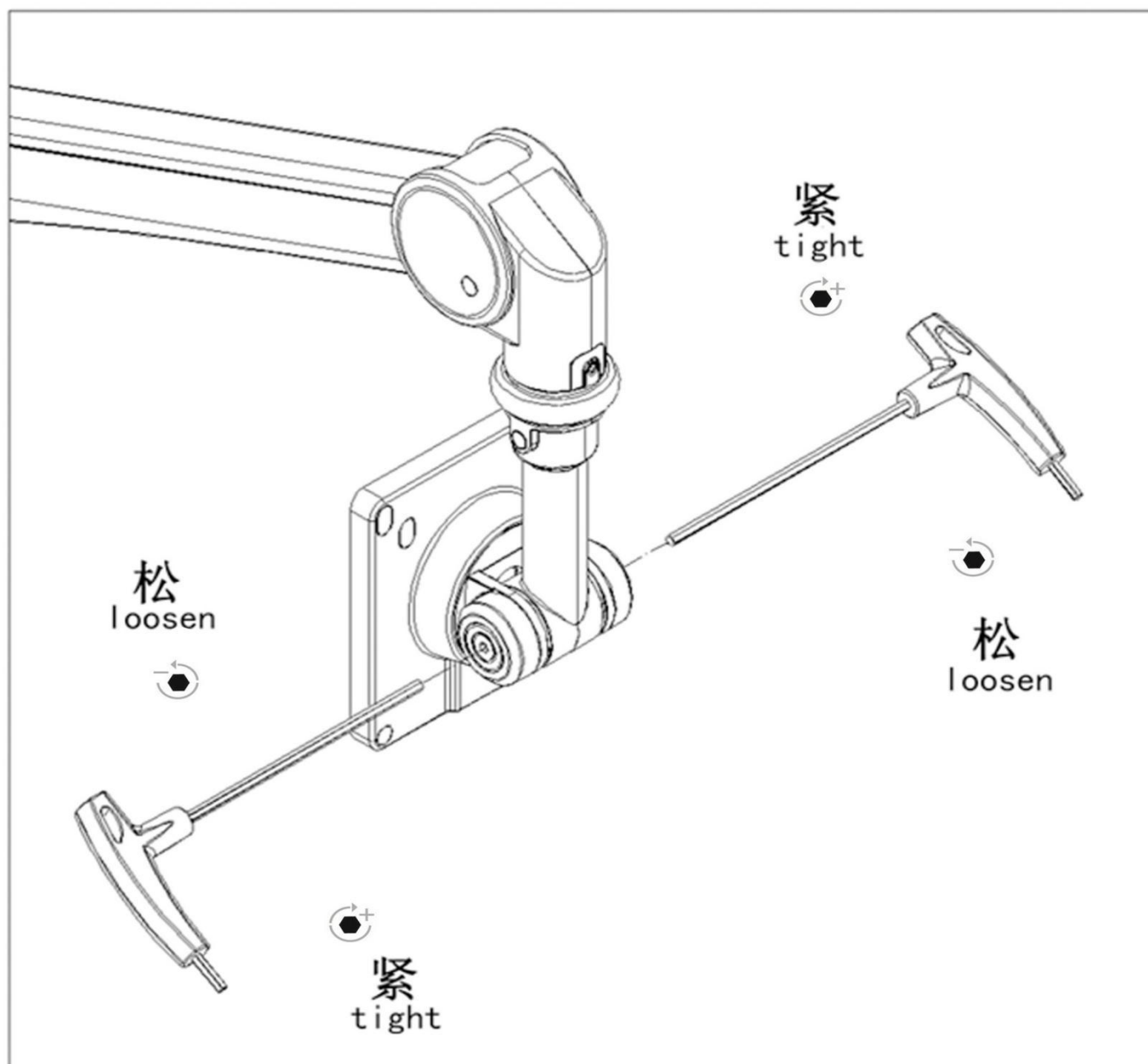## Pitching angle adjustment

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
## Block reduction

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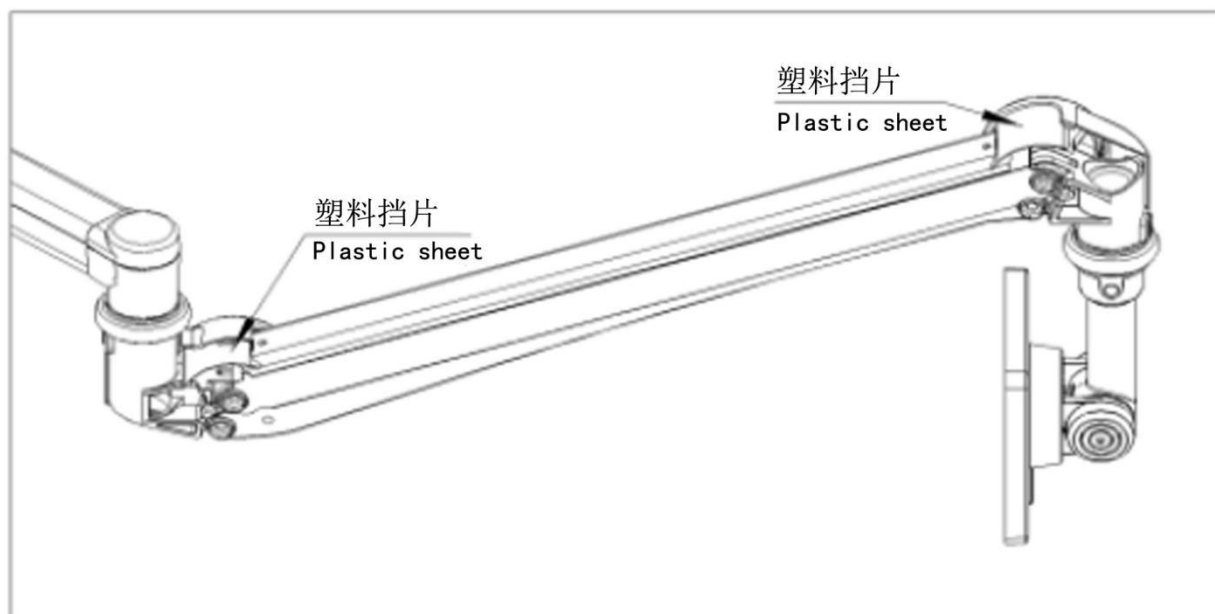
Install it in the following diagram

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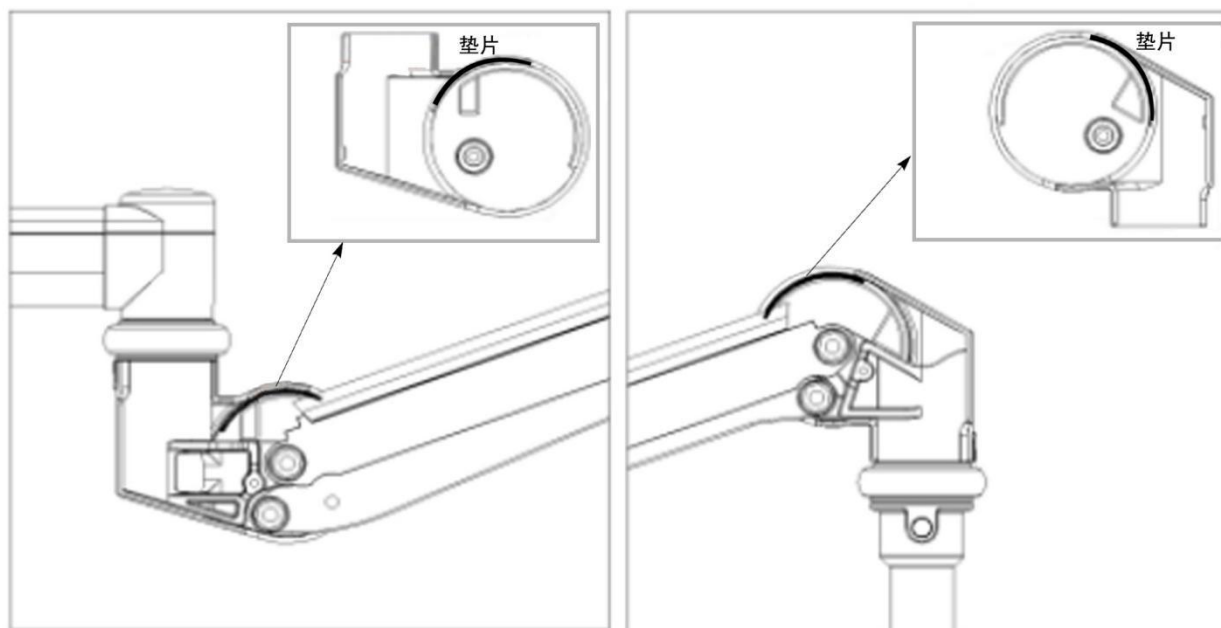The plastic sheet is to be installed in the slot

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